


 YOUR COMMUNITY OF EXCELLENCE	How To Install IntelViewer Guide			
	Document Number	Creation Date	Revision Date	Page 1 of 4
	IT-07171	6/26/2013		

1. Purpose

The purpose of this document is to describe the process required for installing IntelViewer

- a. Read and accept the terms of the IntelViewer license agreement, click Next.
- b. Read and accept the terms of the Java license agreement, and then click Next.
- c. The Choose Install Folder window appears.

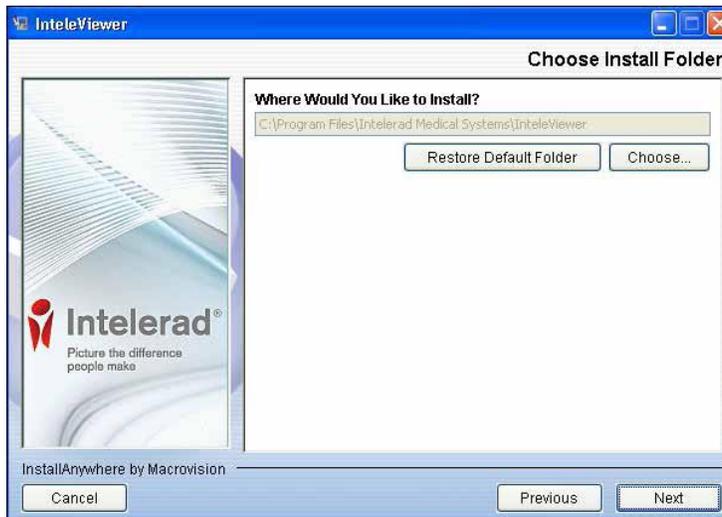


By default, the application will be installed in:

C:\Program Files\Intelrad Medical Systems\IntelViewer

- d. To select a different folder in which to install the application, click Choose and select the folder in the dialog that appears.
- e. Click Next.
- f. Choose which users can automatically upgrade IntelViewer and the Image Fusion module when new versions are available. You can choose All Users or Administrator/Power User Only. If you choose Administrator/Power User Only, only users with the corresponding privileges assigned to their Windows user account will be able to upgrade the application.



- g. Click Next.
- h. Disable the DICOM service. Note: This option appears only if your Windows user account is assigned Administrator privileges.

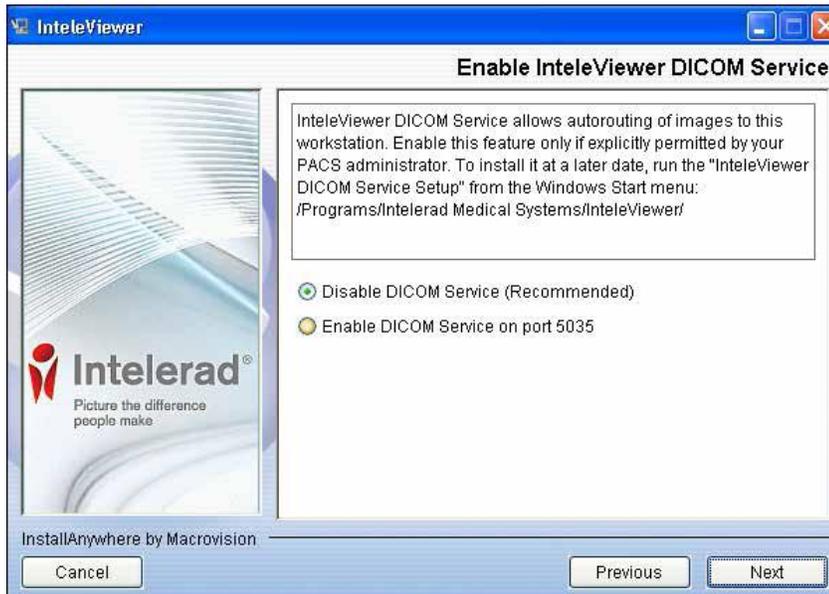


- i. Click Install. The IntelViewer installation is complete. An IntelViewer application icon appears on your Windows desktop and Quick Launch bar.


j. Enter the “Server Bookmark” by clicking on Edit Bookmarks



k. Click on Add

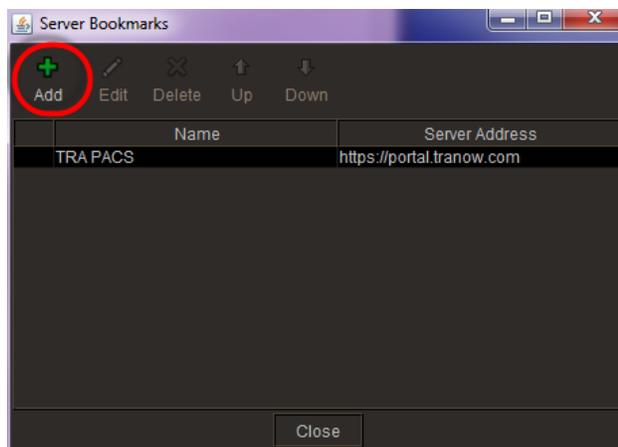

l. Enter Server URL: <https://portal.tranow.com> , select Use Custom Server Name and enter TRA PACS, select OK

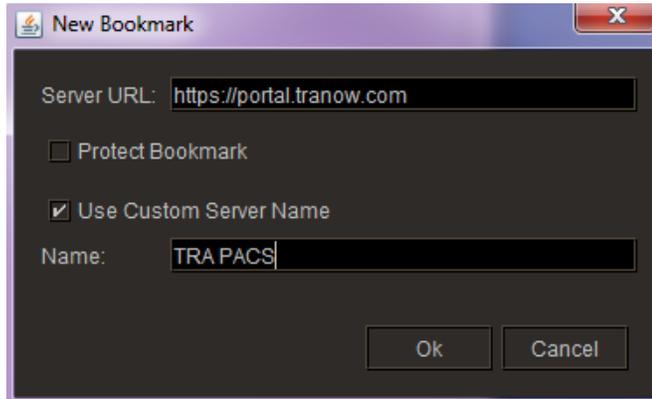

m. Login with appropriate Username and Password



2. Revision History

Revision Date	Description	Revision Purpose	By
6/24/2013	Initial Creation		Cheryl Isley