



InteleViewer

Mac Installation Guide
4.6.1

COPYRIGHT

© 2010-2013 Intelrad Medical Systems Incorporated. All Rights Reserved.

INTELERAD CONFIDENTIAL—NOT FOR DISTRIBUTION

No part of this publication may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying and recording, or by any information storage or retrieval system, without prior written consent from Intelrad Medical Systems Incorporated. If the receiving party of this publication is bound by a NDA, the contents of this publication should be considered "confidential" information.

Except as expressly provided otherwise in writing, the information provided in this document is provided AS IS, without any condition or warranty, whether written, oral, express, implied, legal, or statutory. No condition or warranty of merchantability or fitness for a particular purpose applies to anything provided by Intelrad except as expressly provided in writing within a Service or License Agreement. Without limiting the foregoing, except as specifically set forth in a Service or License Agreement, Intelrad nor any of its suppliers warrants this documentation is or will be accurate, error free or meets or will meet user requirements.

TRADEMARKS

Intelrad and the Intelrad logo, IntelePACS, InteleBrowser, InteleViewer, IntelePass, Reporting Worklist Module, Transcription Module, InteleConnect, and Dictation Module are trademarks of Intelrad Medical Systems Incorporated.

THIRD-PARTY TRADEMARKS

Adobe, Acrobat, and Reader are either registered trademarks or trademarks of Adobe Systems Incorporated in the United States and/or other countries. Sun, Sun Microsystems, and Java are trademarks or registered trademarks of Sun Microsystems, Inc. in the United States and other countries. Intel, Pentium, Pentium II Xeon, and Pentium III Xeon are trademarks or registered trademarks of Intel Corporation or its subsidiaries in the United States and other countries. Microsoft and Windows are either registered trademarks or trademarks of Microsoft Corporation in the United States and/or other countries.

All other brand names, product names, or trademarks belong to their respective holders.

INDICATIONS FOR USE

IntelePACS™ is a device that receives digital images and data from various sources (such as, CT scanners, MR scanners, ultrasound systems, R/F units, computer and direct radiographic devices, secondary capture devices, scanners, imaging gateways, or other imaging sources). Images and data can be communicated, processed, manipulated, enhanced, stored, and displayed within the system and/or across computer networks at distributed locations. Post-processing of the images can be performed using Multi-Planar Reconstruction (MPR).

Lossy compressed mammographic images and digitized film screen images must not be reviewed for primary image interpretations. Mammographic images may only be interpreted using an FDA approved monitor that offers at least 5 Mpixel resolution and meets other technical specifications reviewed and accepted by FDA.

Typical users of this system are trained professionals, physicians, nurses, and technicians.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Intelrad Medical Systems Incorporated
Central Station
895 de la Gauchetiere St W, Suite 400
Montreal, QC Canada H3B 4G1

DECLARATION OF CONFORMITY

We hereby certify that the IntelePACS product, which includes InteleViewer, InteleBrowser, Reporting Worklist Module, and Transcription Module, is a Class IIa Medical Device and is in compliance with Council Directive 93/42/EEC and marked with



AUSTRALIAN SPONSOR

Emergo Asia Pacific
201 Sussex Street, Darling Park, Tower 2, Level 20
Sydney, NSW 2000, Australia
tel: +61.0.2.9006.1662

Title: InteleViewer Mac Installation Guide
Software version: 4.6.1
Date: March 20, 2013
Part number: IVMCEN461IG-O Issue 001



InteleViewer Mac Installation Guide

This guide provides the information you need to install InteleViewer™ on an Apple® Mac OS X® operating system.

In this Guide:

About InteleViewer on a Mac	2
Installing InteleViewer on a Mac	2
Changes to InteleViewer Shortcut Keys on a Mac	3
InteleViewer Feature Limitations on a Mac	3



About IntelViewer on a Mac

You can run IntelViewer sessions on the Mac OS X operating system provided you have the necessary user privilege and system requirements.

User and Site Privileges

You must have the Mac Client user privilege enabled to open an IntelViewer session on a Mac.

The site you want to access must also be configured to allow users with the Mac Client user privilege to run IntelViewer sessions. Unless both conditions are met, you cannot log in to IntelViewer.

System Requirements

To run IntelViewer on a Mac, the following is required:

- IntelPACS 3.7.1 R20, IntelPACS 3.8.1 R14, IntelPACS 4.1.1 R7, or IntelPACS 4.2.1 or later
- Mac OS X[®] Snow Leopard[™] or OS X[®] Mountain Lion with 64-bit processor
- Java[®] 1.6 64-bit

Installing IntelViewer on a Mac

You can run IntelViewer on a Mac.

To install IntelViewer on a Mac:

1. Log in to IntelBrowser.
2. In the IntelBrowser main menu, click Installers.
The Intelrad User Resources page appears.
3. Click IntelViewer Tracks.
The IntelViewer Tracks page appears.
4. From Evaluation Track, Advanced Track, or Stable Track, click Download IntelViewer for the Mac OS X Installer.
The Opening IntelViewer.dmg dialog appears.
5. Enable the Save File option and click OK.
The Downloads dialog appears.
6. Double-click *IntelViewer.dmg*.
This disk image contains the IntelViewer application and a license agreement.

7. Drag the IntelViewer application to the Applications folder at the top level of the hard disk on your Mac.
8. Double-click IntelViewer.
9. To provide quick access to IntelViewer, add it to the Dock.

Changes to IntelViewer Shortcut Keys on a Mac

On a Mac keyboard, the shortcut keys for performing the actions below are different than those on a Windows® PC keyboard:

- To access Help documentation, press **FN+F1**.
- To reset an image, press **CTRL+DELETE**.
- To delete a selected measurement, press **FN+DELETE**.

IntelViewer Feature Limitations on a Mac

When running IntelViewer on a Mac, the following features are disabled or unavailable:

- Viewing the worklist.
- Exporting to DICOM by using IntelViewer.
- Burning to CD-ROM.
- Integrations with third-party applications running on Windows®.
- Philips® SpeechMike™ integration.
- Launching IntelViewer from IntelBrowser.
- Advanced Visualization, except Orthogonal MPR (which is available).
Note: Your user account must be assigned the Image Reformat Functionality privilege to use the Orthogonal MPR tool on a Mac.
- To Sign tab.
- Upgrading by using the Utilities | Check for Application Update command.
- Terminal services.
- DICOM services, including autorouting, retrieval, working offline, sending to PACS.
- Multi-Method Reporting.
- Exporting images to video.